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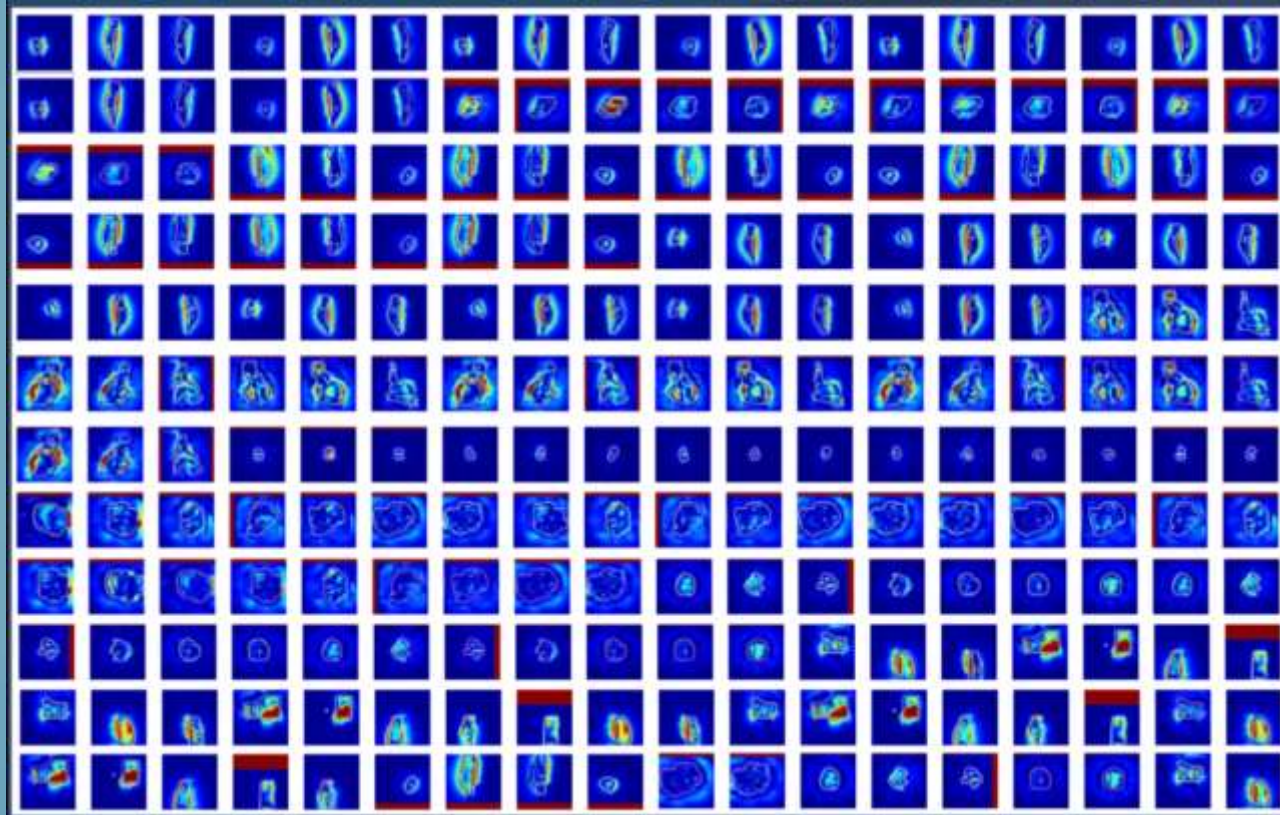
Presented by: Wolfram Laub, PhD, MBA, Director of Medical Physics
Date: January 2012

MEDINEX
Medical Innovation Excellence

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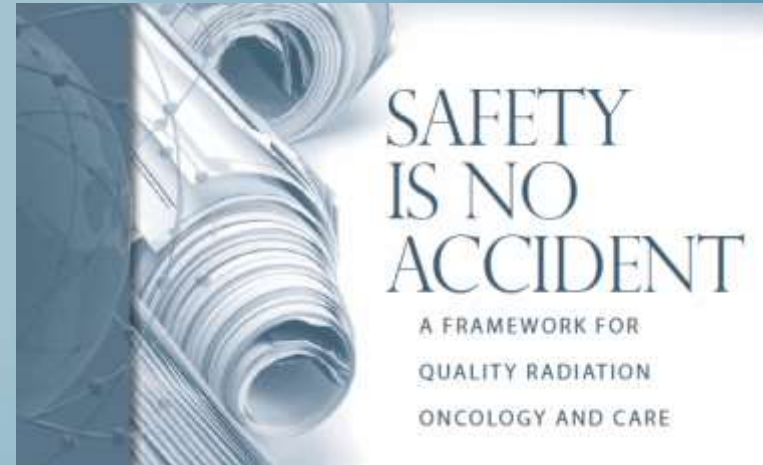
How safe is safe: Risk in radiotherapy (RT)

- Rate of misadministration in RT is 0.2 % (1 in 600)
- Rate of serious injury: airline accidents is 1 in 10 million
➢ 16,000 times lower than that of RT
- Rate of serious injury in RT is 1000 times higher
- In reality no one knows
- Risk profiles of anesthesia: similar to airline industry

Ford and Terezakis; Red Journal 78, 321-322, 2010

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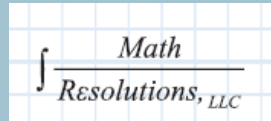
MEDINEX
Medical Innovation Excellence

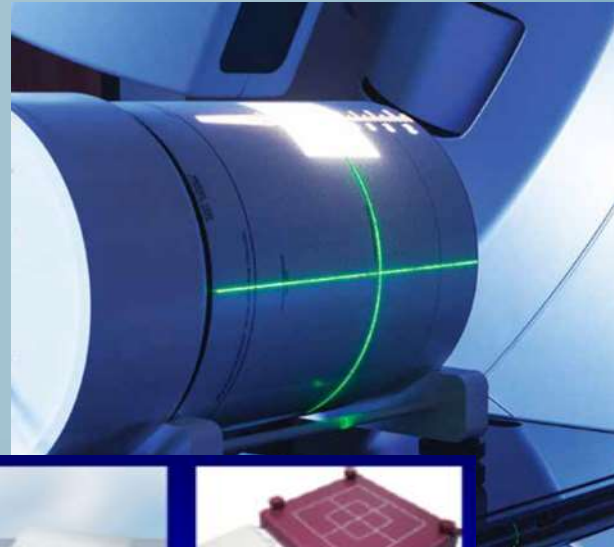
MEDINEX PRODUKTPALETTE PARTNER

1. DosimetryCheck
2. Mobius 3D
3. MobiusFX
4. FractionLab
5. DoseLab
6. EQA2
7. ADQ
8. FilmQAPro
9. OnQrts
10. ImSimQA

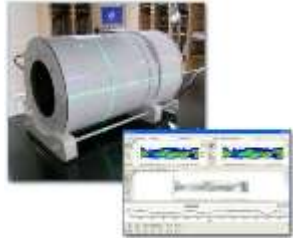
MEDINEX
Medical Innovation Excellence

QS IMRT - STAND DER TECHNIK

MEDINEX
Medical Innovation Excellence

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Hardware-Based

30-90 MINUTES

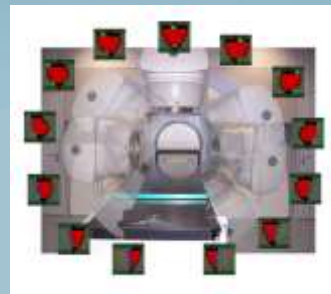
EPID

EPID

EPID

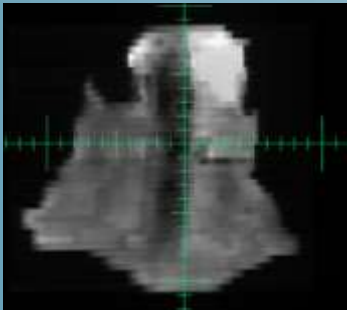
EPID

EPID

EPID

EPID

EPID

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EPID

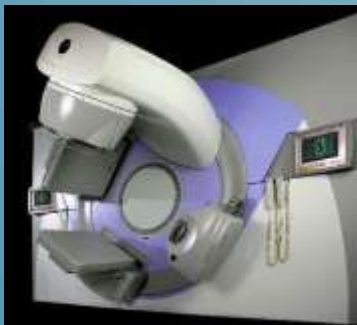
EPID

EPID

EPID

A literature review of electronic portal imaging for radiotherapy dosimetry

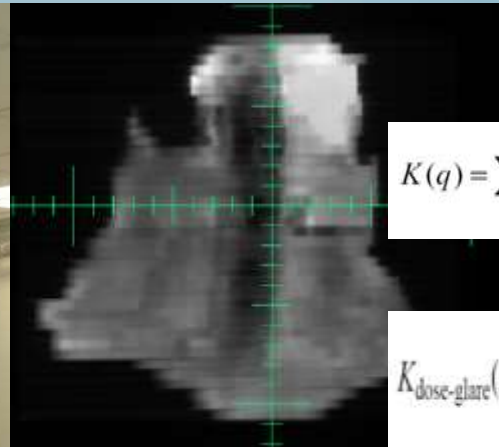
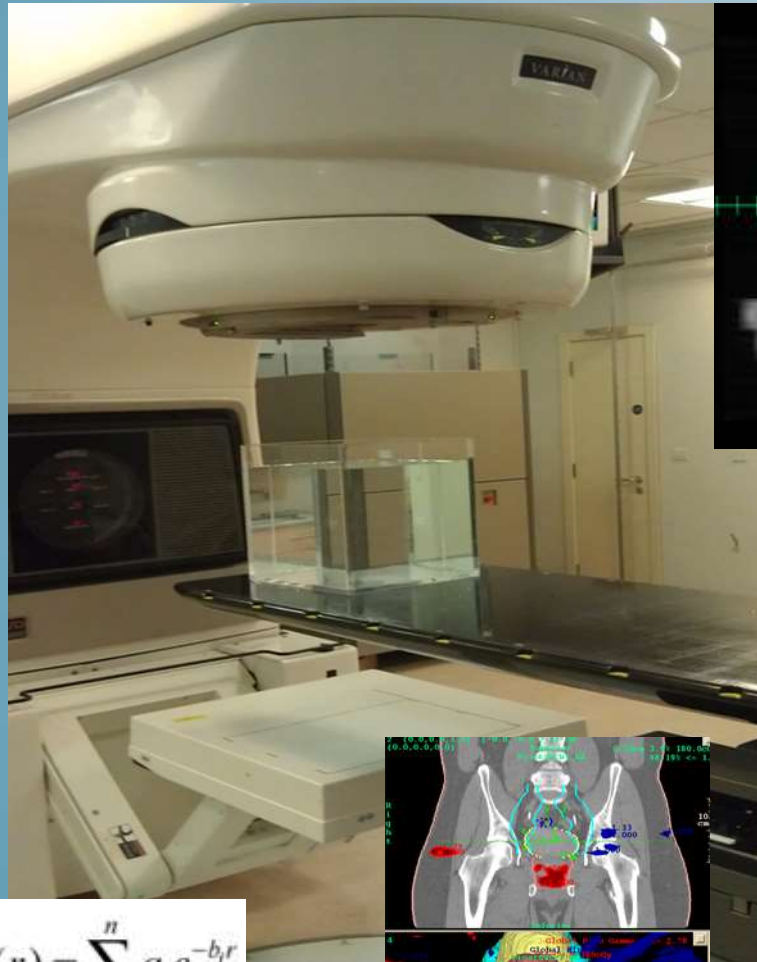
Wouter van Elmpt^{a,*1}, Leah McDermott^{b,1}, Sebastiaan Nijsten^a, Markus Wendling^b, Philippe Lambin^a, Ben Mijnheer^{a,b}

MEDINEX
Medical Innovation Excellence

DOSIMETRYCHECK

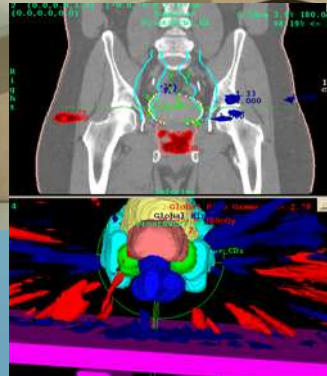
EPID 3D IN VIVO DOSIMETRIE

$$K(q) = \sum_i^n a_i \frac{2\pi b_i}{(4\pi^2 q^2 + b_i^2)^{3/2}}$$

$$K_{\text{dose-glare}}(r) = \exp(-a_1 r) + a_2 \exp(-a_3 r) + a_4 \exp(-a_5 r) + a_6 \exp(-a_7 r) + a_8 \exp(-a_9 r). \quad (7)$$

$$k(r) = \sum_i^n a_i e^{-b_i r}$$

JOURNAL OF APPLIED CLINICAL MEDICAL PHYSICS, VOLUME 6, NUMBER 4, FALL 2005

A method for deconvolution of integrated electronic portal images to obtain incident fluence for dose reconstruction

Wendel Dean Renner,¹ Kevin Norton,² and Timothy Holmes³

Math Resolutions,¹ LLC, 5975 Gales Lane, Columbia, Maryland 21045-3841; Hartford Hospital,² Dept. Radiation Oncology, 80 Seymour Street, Hartford, Connecticut 06102 and The University of Connecticut Health Center, Dept. Radiation Oncology, Room CG178, 263 Farmington Avenue, Farmington, Connecticut 06030-2930; St. Agnes Health Care,³ Dept. Radiation Oncology, 900 Caton Avenue, Baltimore, Maryland 21229 U.S.A. WendelDRenner@mathresolutions.com, knorton@uchc.edu, tholmes@stagnes.org

Received 19 January 2005; accepted 13 July 2005

MEDINEX
Medical Innovation Excellence

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(12) **United States Patent**
Renner (10) Patent No.: **US 8,605,857 B1**
(45) Date of Patent: ***Dec. 10, 2013**

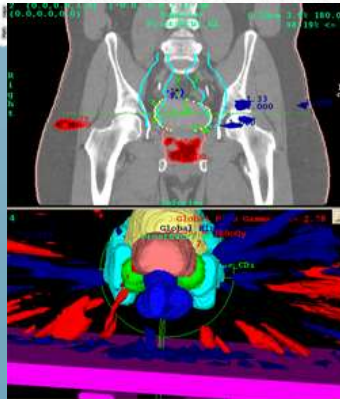
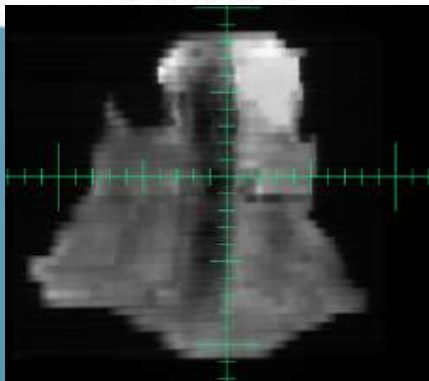
(54) METHOD AND SYSTEM TO RECONSTRUCT TREATMENT DOSE TO A PATIENT FROM INTEGRATED EXIT-TRANSIT IMAGES OF RADIATION FIELDS TAKEN DURING TREATMENT (56) **References Cited**
U.S. PATENT DOCUMENTS
5,754,622 A * 5/1998 Hughes 378/65
6,438,202 B1 * 8/2002 Olivera et al 378/65
6,636,622 B2 * 10/2003 Mackie et al 382/132

(12) **United States Patent**
Renner (10) Patent No.: **US 8,351,572 B1**
(45) Date of Patent: **Jan. 8, 2013**

(54) METHOD AND SYSTEM TO RECONSTRUCT TREATMENT DOSE TO A PATIENT FROM INTEGRATED EXIT-TRANSIT IMAGES OF RADIATION FIELDS TAKEN DURING TREATMENT (56) **OTHER PUBLICATIONS**
Wendell Dean Renner, Kevin Norton and Timothy Holmes, A Method for Deconvolution of Integrated Electronic Portal Images to Obtain Incident Fluence for Dose Reconstruction, *Journal of Applied Clinical Medical Physics*, vol. 6, No. 4, Fall 2005, pp. 22-39.
Markus Wendling, Leah N. McDermott, Anton Mans, Jan-Jacob Sonke, Marcel van Herk, and Bea J. Mignhofer, A Simple Backprojection Algorithm for 3D in vivo EPID Dosimetry of IMRT Treatments,

(12) **United States Patent**
Renner (10) Patent No.: **US 6,853,702 B2**
(45) Date of Patent: **Feb. 8, 2005**

(54) RADIATION THERAPY DOSIMETRY QUALITY CONTROL PROCESS (56) **References Cited**
U.S. PATENT DOCUMENTS
5,754,622 A * 5/1998 Hughes 378/65
6,038,283 A * 3/2000 Carol et al 378/65
6,038,284 A * 3/2000 Hernandez-Guerra et al 378/65
6,175,761 (57) **Abstract**
6,438,202 (57) **Abstract**

$$K(q) = \sum_i^n a_i \frac{2\pi b_i}{(4\pi^2 q^2 + b_i^2)^{3/2}}$$

$$k(r) = \sum_i^n a_i e^{-b_i r}$$

$$K_{\text{dose-glare}}(r) = \exp(-a_1 r) + a_2 \exp(-a_3 r) + a_4 \exp(-a_5 r) + a_6 \exp(-a_7 r) + a_8 \exp(-a_9 r). \quad (7)$$

JOURNAL OF APPLIED CLINICAL MEDICAL PHYSICS, VOLUME 6, NUMBER 4, FALL 2005

A method for deconvolution of integrated electronic portal images to obtain incident fluence for dose reconstruction

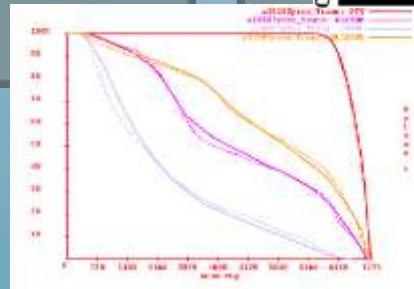
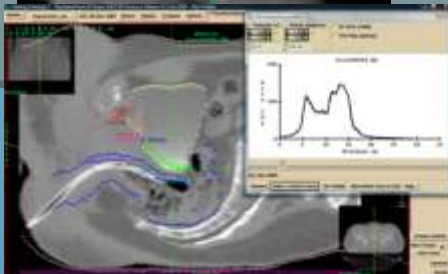
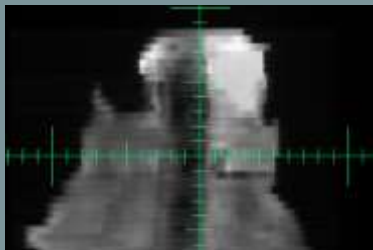
Wendel Dean Renner,¹ Kevin Norton,² and Timothy Holmes³
Math Resolutions,¹ LLC, 5975 Gales Lane, Columbia, Maryland 21045-3841; Hartford Hospital,² Dept. Radiation Oncology, 80 Seymour Street, Hartford, Connecticut 06102 and The University of Connecticut Health Center, Dept. Radiation Oncology, Room CG178, 263 Farmington Avenue, Farmington, Connecticut 06030-2930; St. Agnes Health Care,³ Dept. Radiation Oncology, 900 Caton Avenue, Baltimore, Maryland 21229 U.S.A.
WendelDRenner@mathresolutions.com, knorton@uhc.edu, tholmes@stagnes.org

Received 19 January 2005; accepted 13 July 2005

MEDINEX
Medical Innovation Excellence

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MEDINEX
Medical Innovation Excellence

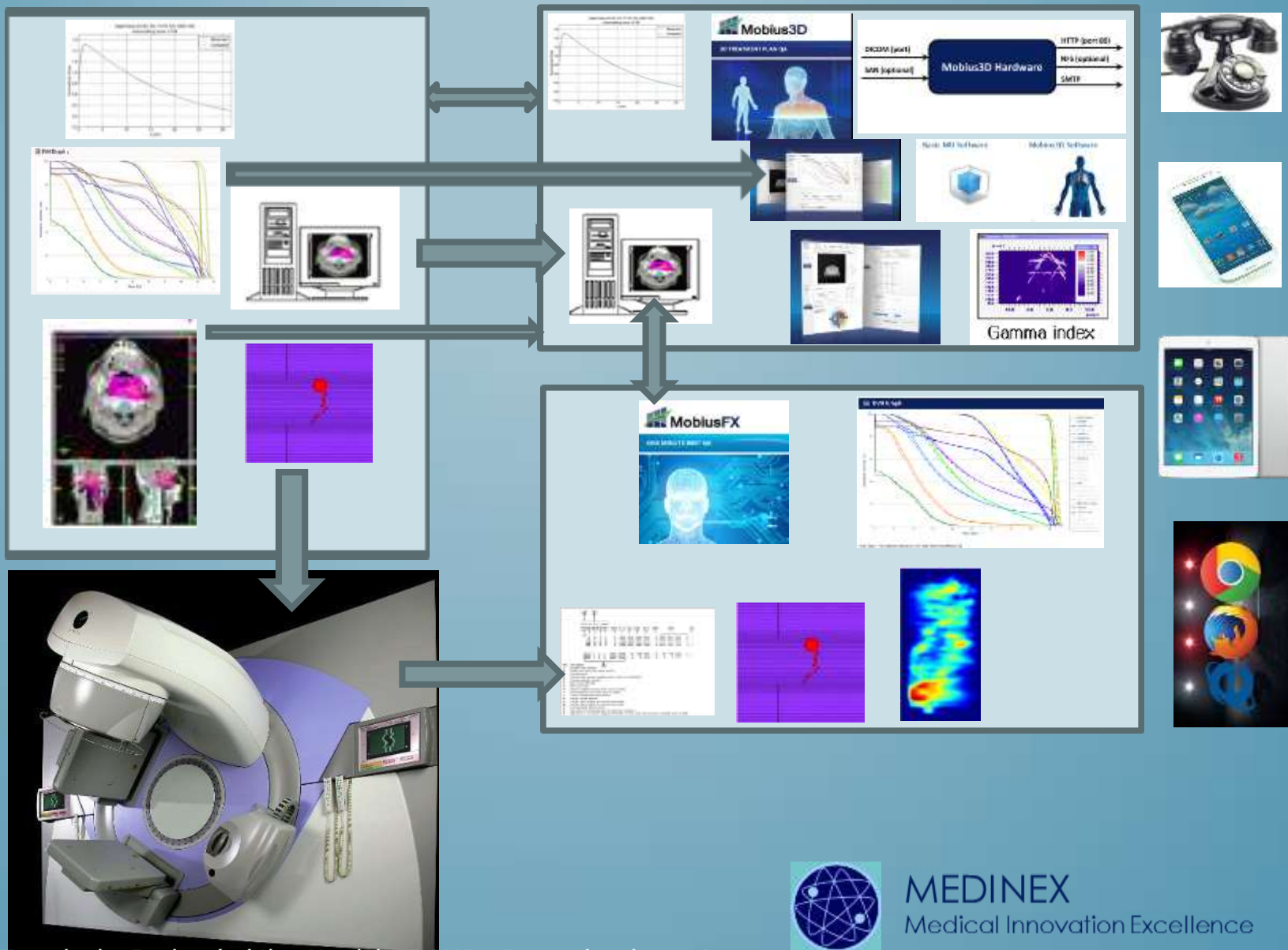
DOSIMETRYCHECK

- 1 NO EQUIPMENT SET UP REQUIRED FOR PRE-TREATMENT QA
- 2 TRANSIT IN-VIVO and PRE-TREATMENT PATIENT DOSIMETRY
- 3 INDEPENDENT MEASURED DOSE CALCULATIONS with & without the patient in the Beam
- 4 PROVIDES 2-D and 3-D ANATOMICAL DOSE RESULTS ANALYSES of:
- 5 PRE TREATMENT and ON TREATMENT IN-VIVO MEASUREMENTS of PATIENTS
- 6 MEASURED DOSE VERIFICATION USING PATIENT CT DATA SET
- 7 WORKS WITH: ANY RTP SYSTEM
- 8 STRUCTURE ANALYSIS: 3D DOSE VOLUME HISTOGRAMS (DVHs) &
- 9 GAMMA VOLUME HISTOGRAMS (GVHs)
- 10 3D PREDICTED (PLANNED) VS. ABSOLUTE DOSE (MEASURED) ANALYSIS
- 11 SUPPORTS: Conventional IMRT, VMAT, RapidArc and MORE
- 12 WORKS WITH: Elekta iView Portal Imager, Varian's Portal Vision, R and E Arm Portal Imager's and Siemens Optiview Portal Imager

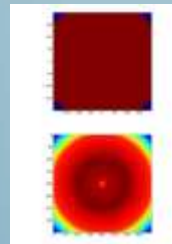
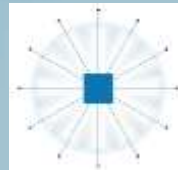
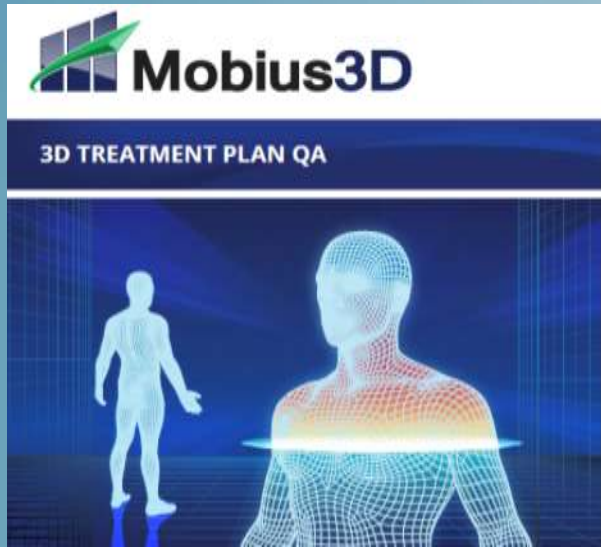
MEDINEX
Medical Innovation Excellence

MOBIUS 3D UND MOBIUS FX

MEDINEX
Medical Innovation Excellence

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MEDINEX
Medical Innovation Excellence

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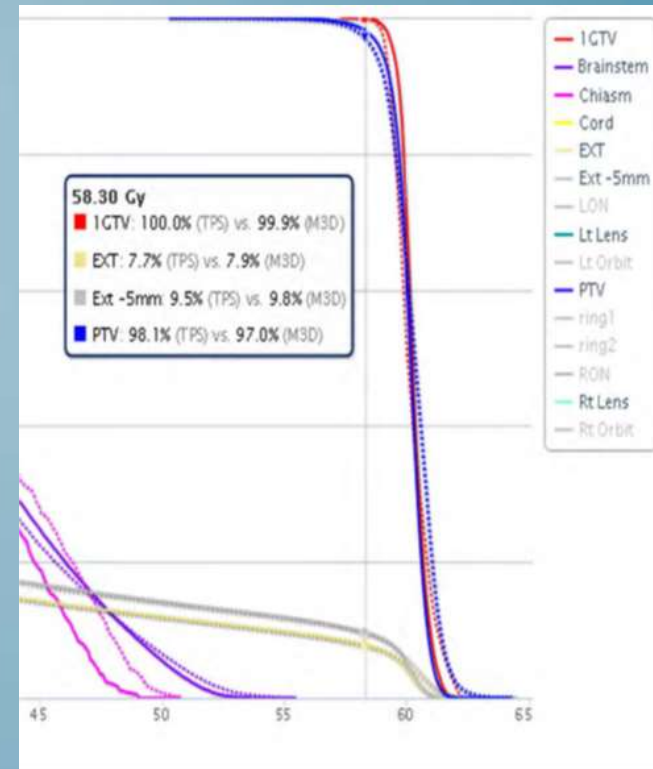
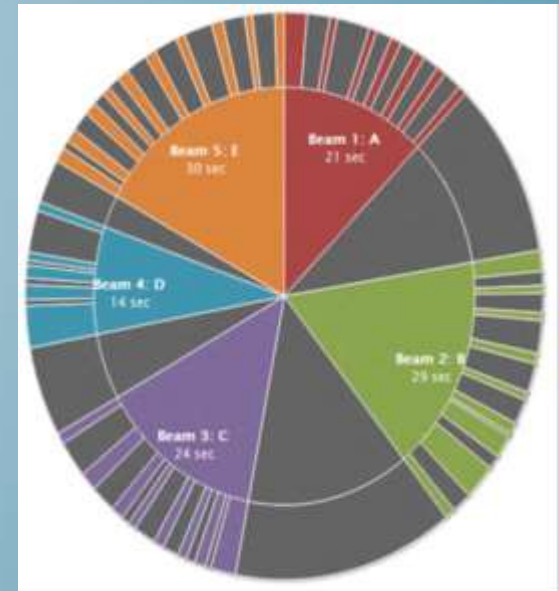
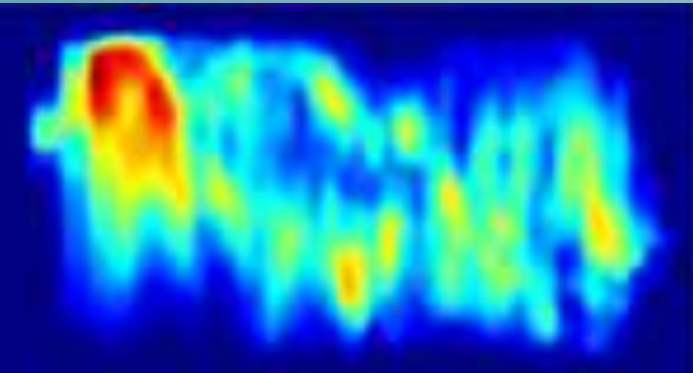
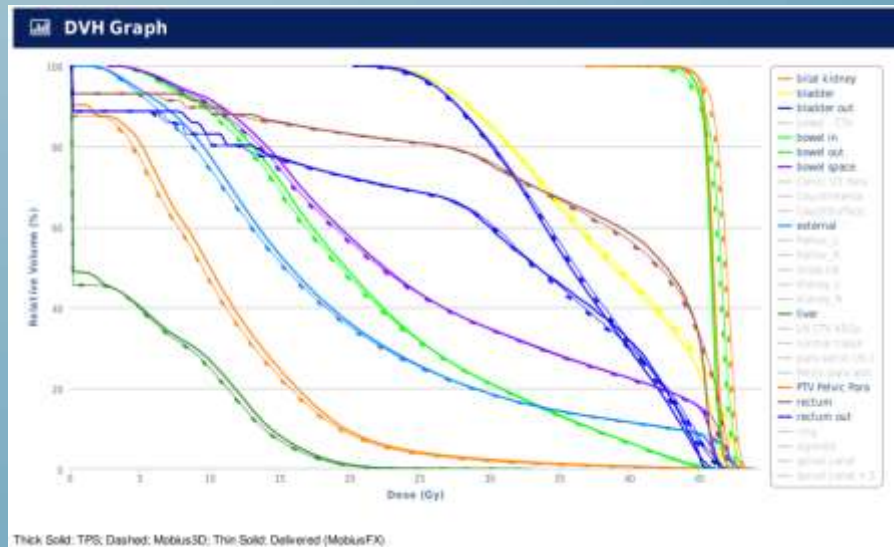
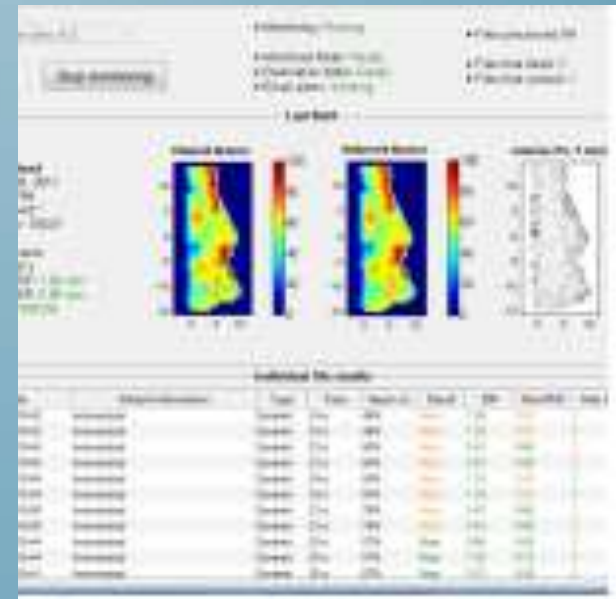
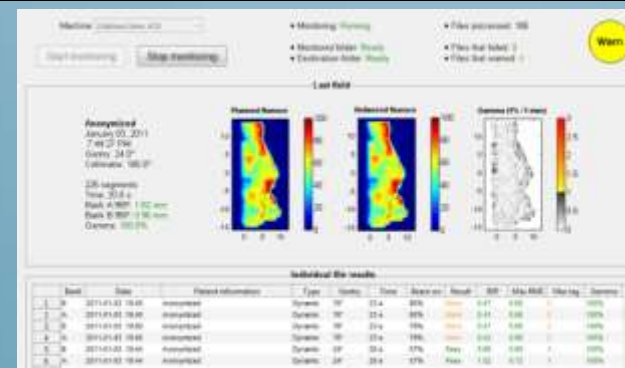
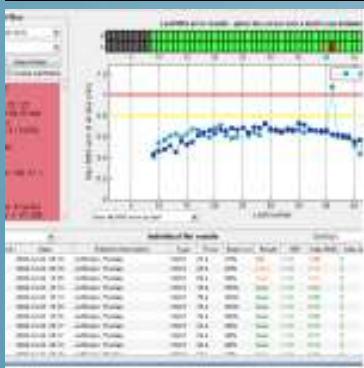
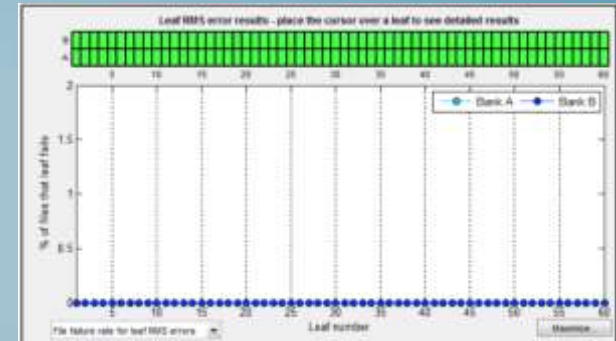
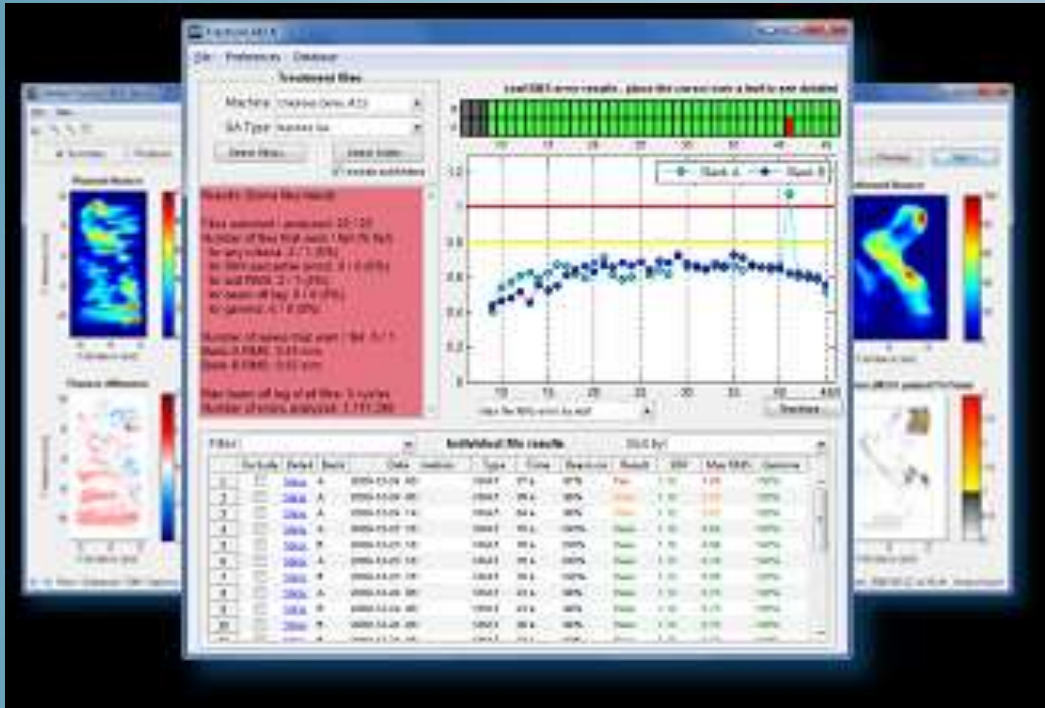
FRACTION LAB

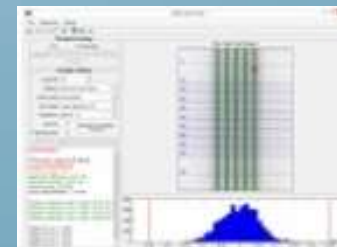
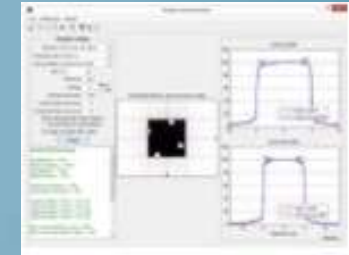
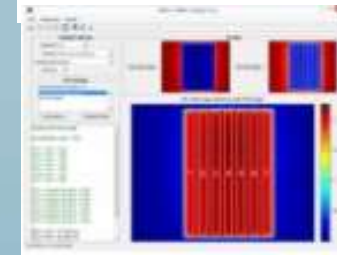
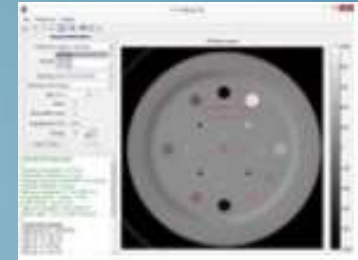
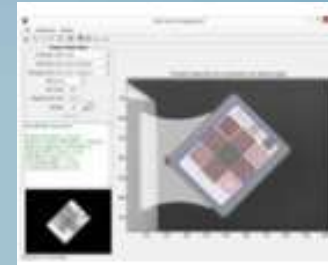
MEDINEX
Medical Innovation Excellence

DOSELAB

POWERFUL TG-142 QA

MEDINEX
Medical Innovation Excellence

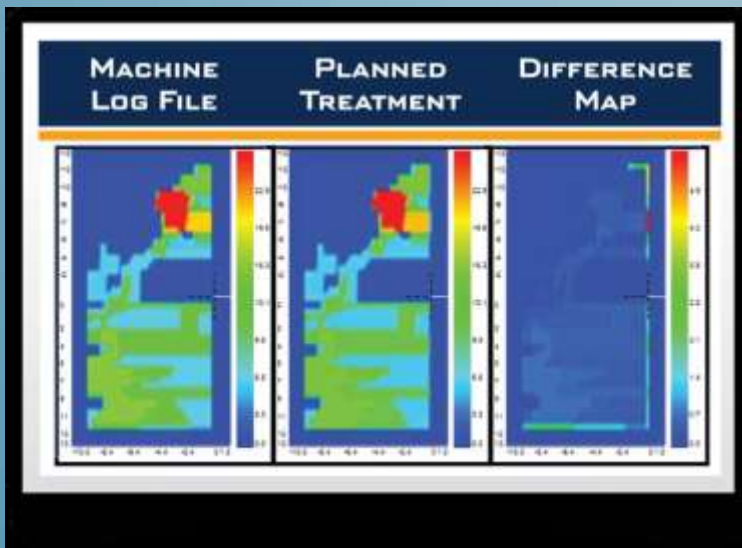
ADQ

ADQ

Key Features

- Automated comparison of machine log files to planned deliveries
- Administrative reports document delivery QA every day... automatically
- Compatible with **Varian Dynalog** and TrueBeam Trajectory log files
- Simple and Intuitive web-based user interface
- Create user accounts and manage access privileges
- Audit feature monitors and tracks all configuration changes
- No external hardware required
- **SQL database for archiving results**
- Automated message center for emailing comparison and summary reports
- **Produce daily, weekly and monthly summary reports**
- Generate on-demand administrative and comparison reports
- Configure Pass/Warning/Fail tolerances

MEDINEX
Medical Innovation Excellence

EQA2

Key Features

- Analyzes TG142 recommended Machine QA tests automatically
- Create custom QA forms to archive and trend data
- Perform Trend Analysis to track and identify patterns in QA
- Image analysis runs unattended in the background
- Supports multiple user levels for controlled access and security
- Flexible software architecture enables distributed use and remote access
- Receive detailed up-to-date activity across the network from the QUASAR™ Message Centre
- Organize your QA process with the QA scheduler
- Supports CITRIX-enabled thin clients with unlimited installations

MEDINEX
Medical Innovation Excellence

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